The background of the slide is a detailed, grayscale image of a leaf's venation system. It shows a network of veins, with several prominent, thick primary veins running diagonally across the frame. A dense, intricate web of smaller secondary and tertiary veins fills the spaces between the primary veins, creating a complex, honeycomb-like pattern. The overall appearance is that of a delicate, organic structure.

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The main reason for indicating the totally implanted catheter is to offer the patient a permanent vascular access device, regardless of age, without puncturing peripheral blood vessels. The Life-Port Vascular Access System is indicated for patients who require repeated vascular access for the infusion of medication, fluids, blood products and nutrients. Principally for cases where the solutions utilized present an osmolality > 600 osm. The major benefit is avoiding phlebitis and thrombosis, providing comfort throughout treatment.

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The implantation of the Life-Port Vascular Access System is contraindicated in cases of acute infection, sepsis, bacteremia, peritonitis, endocarditis; as well as in cases of incompatibility with any of the component materials of the system.

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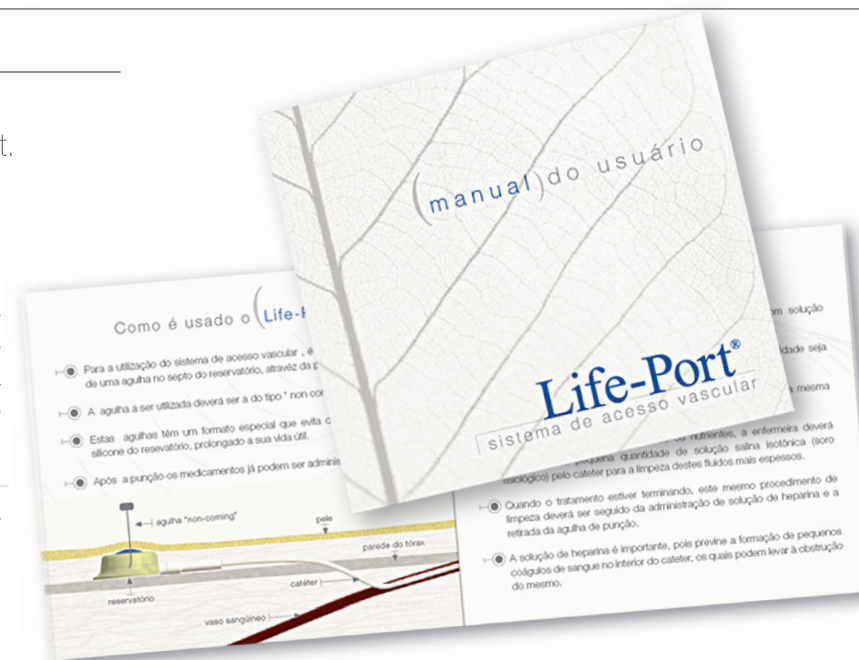
* Material in accordance with the ASTM (America Society for Testing and Materials) Norm for use in implants.

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The Life-Port Vascular Access Systems come with a Patient's Guide developed exclusively for the person using the Life-Port. It includes information regarding implantation, its use, care and frequently asked questions. All of the support and information that your patient needs can be found in this Guide that should be given to the patient.

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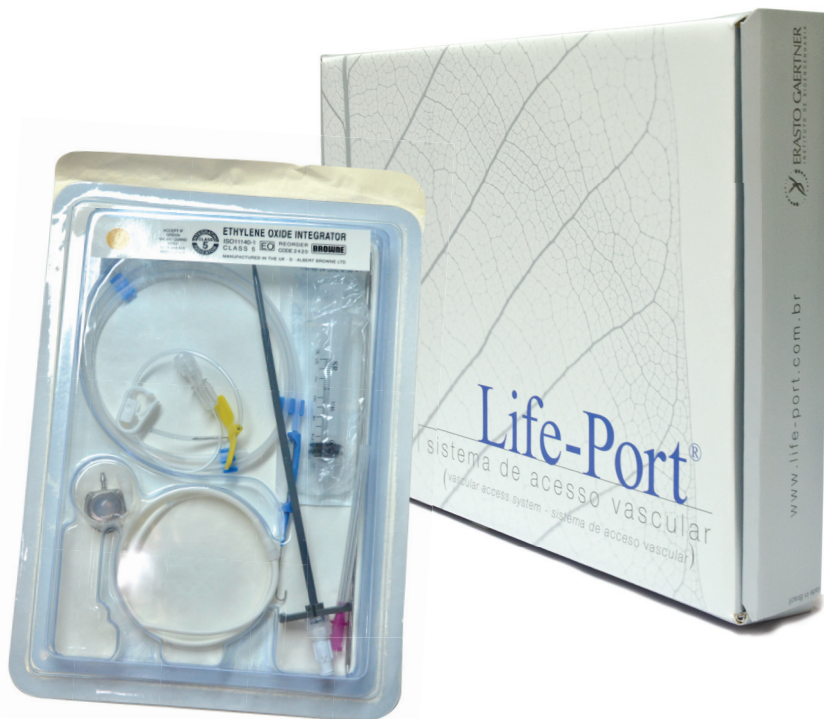
This card identifies the Life-Port user with information on the use of the Life-Port.

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The Life-Port On-line Portal on the internet provides all of the necessary information for ensuring the quality and success of treatment using Life-Port. There are different access areas for doctors, nurses and the patient, with specific information provided for each one.

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Customer Service offers the support of specialized professionals including doctors, nurses, pharmacists and engineers ready to answer your doubts regarding the Life-Port Vascular Access Systems. Phone: 55 41 3361-5210, 55 41 3361 5212, www.ibeg.ind.br, ibegcomercial@erastogaertner.com.br

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For specific information regarding the contraindications, warnings, precautions and potential complications, consult the guide enclosed with each system.

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